


## INSTRUCTIONS FOR THE MULTICHAIR 3000



MULTICHAIR 3000



SEAT FRAME WITH ONE LEG  
INSTALLED

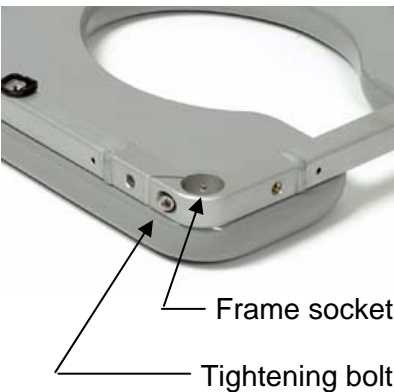

CLOSE-UP OF SEAT FRAME

**NOTICE:** Please see section **3: LIMITED WARRANTY** of these instructions. If warranty is not acceptable, return the chair for a full refund. Contact Nuprodx, Inc. for shipping instructions.

### 1: SETUP INSTRUCTIONS:

- a. Unpack all parts from the shipping carton. Remove all tape and/or packing from the chair. Keep shipping materials for future use. Note the location of padding materials and sheet plastic which protects parts of your chair.
- b. Position the seat frame upside down and install the four legs. Note that the top of each leg has a machined portion approximately one inch wide. The top end of each leg should fit easily into the socket of the frame. Make sure that the legs slide "home" into the sockets. Use the included  $\frac{1}{4}$ " Allen wrench to tighten the bolts located at the corners of the frame. For ease of future leg height adjustment, position each leg in the socket so that the adjustment holes face outward.
- c. Turn the chair right side up and install the backrest assembly. It has two threaded bolts attached to the back frame tubes. Position the backrest with both bolts installed loosely in the frame. Start tightening the bolts by hand before using the  $\frac{1}{4}$ " Allen wrench.

### 2: OPERATING INSTRUCTIONS:

- a. Leg Adjustment: When you adjust the legs, make sure that the brass buttons are fully seated in the legs. Note: if you push too hard the button could become stuck inside the inner leg.
- b. Seat and back cushions. The cushions are quality closed-cell foam, designed to provide comfort and protection. Make sure that you don't pack anything on top of it (like the back assembly, arm rests, or any tools). Because it doesn't have air bubbles, the foam will not spring back after being compressed for a long time (like overnight on an airplane trip.) Any dents in the foam will not spring back and are permanent. Also, do not put anything sharp on top of the cushions.
- c. Check the tightness of all bolts each time that you use the chair.
- d. Periodically apply light oil to the threads of the bolts.

### 3: LIMITED WARRANTY

- a. Please read this warranty before operating or using your Multichair 3000. By operating or using the chair, you agree to the terms of this warranty. If you do not agree, return the chair to Nuprodx, Inc. and you will receive a full refund of the price you paid.
- b. WARRANTY: Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a 30-day unconditional money-back warranty. During that period the chair can be returned for a full refund minus a 10% restocking charge. After that period, there is a one-year limited warranty for all parts except cushions. Because of the fragile nature of the foam, there is no warranty for the cushions.
- c. The warranty period begins on the date you receive the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.
- d. Please do not return the chair to Nuprodx, Inc., without the advance authorization of Nuprodx. Freight charges to the factory are at your expense, unless otherwise authorized by Nuprodx, Inc.
- e. DISCLAIMER: Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.
- f. WARRANTY LIMITATIONS AND EXCLUSIONS: The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc.
- g. This warranty gives you specific legal rights, and you may have other rights that may vary from state to state. Some states do not allow purchase price of the chair.
- h. This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.