

INSTRUCTIONS FOR THE MULTICHAIR 6000

IMPORTANT - READ THIS CAREFULLY AND COMPLETELY!

WARRANTY NOTICE

Please see LIMITED WARRANTY on the last page of these instructions. If the warranty is not acceptable, return the chair for a full refund. Contact Nuprodx, Inc. for shipping instructions. This chair is designed and intended for indoor shower and bathroom use. It is not designed for outdoor use and will void the warranty if used in this fashion. If thresholds, shower curbs or barriers are encountered that exceed ½" in height, the area in question should be brought within the code for disability accessibility.



MULTICHAIR 6000 - SHOWN WITH TUB ON USERS LEFT
(FOR OPPOSITE HAND REVERSE LEGS)

SET-UP INSTRUCTIONS

- Unpack all parts from the shipping carton. Remove all tape and/or packing from the chair. Keep shipping materials for future use. Note the location of padding materials and sheet-plastic which protects parts of your chair.
- The multiCHAIR 6000 is composed of three pieces: the roll-in section, the bridge section, and the tub section.
- Both the roll-in section and the tub section have four legs. Offset legs with 5" casters for the roll-in section and straight legs with swivel adjustment feet on the tub section.
- Position the frames upside down and install four legs in each frame. Note: The frames are identical except for the location of the locking brackets: the roll-in section has the locking bracket at the right rear corner (when sitting on the chair) and the tub section has the locking bracket at the left rear). If you want the tub to be on your right (when sitting on the chair) reverse the legs.
- Position the roll-in legs so that the offset points out at a 45-degree angle. Note that the top portion of the offset legs have a 1" offset or elbow welded to the end of the leg. Make sure that the legs

slide “home” into the socket. Use the included $\frac{1}{4}$ ” Allen wrench to securely tighten the bolts located at the corners of the frame.

- f. Turn over the frames and install the footrest, back, and arms.
- g. Install the footrest assembly: 1) Slide each swing-away footrest into the footrest clamp already installed on the front legs. 2) Adjust the height and width as required by the user’s feet. 3) Tighten the clamps with the $\frac{1}{4}$ ” Allen wrench.
- h. Install the backrest: 1) Place the backrest against the frame and tighten the two threaded bolts
- i. Install the arms: 1) First, insert the rear end portion of the arm into the slotted tube attached to the backrest. 2) Second, Lower the arm down onto the black ball attached to the side of the seat frame. Insure that the metal latch engages around the black ball

OPERATING INSTRUCTIONS:

- a. Lock the brakes on the casters by pushing down on the rear most textured part of the caster brake levers. Release the brakes on the casters by pushing down on the front textured part of the caster brake levers. Note: ALWAYS LOCK THE BRAKES USING THE CHAIR.
- b. Adjust the height of the bridge and tub section so that there is no more than a $\frac{1}{4}$ ” difference between the height of the roll-in section and the tub section. Use a ruler or straight edge to check this dimension.
- c. Foot Rest Adjustment: Adjust the height and width as required by the user’s feet. Tighten the clamps with the $\frac{1}{4}$ ”Allen wrench.
- d. Leg Adjustment: When you adjust the legs, make sure that the brass buttons are fully seated in the legs. Note: The swivel feet in the tub section have a $\frac{3}{4}$ ” adjustment range. Make big adjustments with the brass buttons and fine adjustments with the swivel feet. ALWAYS CHECK THE SWIVEL FEET BEFORE USING THE CHAIR.
- e. Seat and back cushions. The cushions are quality closed-cell foam, designed to provide comfort and protection. Make sure that you don’t pack anything on top of it (like the back assembly, arm rests, or any tools). Because it doesn’t have air bubbles, the foam will not spring back after being compressed for a long time (like overnight on an airplane trip.) Any dents in the foam will not spring back and are permanent. Also, do not put anything sharp on top of the cushions.
- f. Check the tightness of all bolts each time that you use the chair.
- g. Periodically apply light oil to the threads of the bolts.



CLOSE-UP SHOWING
RULER WITH $\frac{1}{4}$ ” GAP



CLOSE-UP OF BRIDGE
SECTION



CLOSE-UP OF SWING-
AWAY FOOTREST

LIMITED WARRANTY

- a. NOTICE: Please read this before operating or using your Multichair 6000. By operating or using the chair, you agree to the terms of this warranty. If you do not agree, return the chair to Nuprodx, Inc. and you will receive a full refund of the price you paid.
- b. WARRANTY: Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a 30-day unconditional money-back warranty. If you are unhappy, for any reason, with the chair you can return it for a full refund minus a 10% restocking charge. After that period, there is a one-year limited warranty for all parts except cushions. Because of the fragile nature of the foam, there is no warranty for the cushions.
- c. The warranty period begins on the date you received the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.
- d. Please do not return the chair to Nuprodx, Inc., without the advance authorization of Nuprodx. Freight charges to the factory are at your expense, unless otherwise authorized by Nuprodx, Inc.

DISCLAIMER

Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.

WARRANTY LIMITATIONS AND EXCLUSIONS

- a. The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc.
- b. This warranty gives you specific legal rights, and you may have other rights that may vary from state to state.
- c. This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.